

Virtuo Vivo - Import Implant Kits

Virtuo Vivo - Import Implant Kits

Nota Bene: Make sure the software version is the latest.


Nota Bene 2: Be aware for Third-Party implant manufacturers, the scanbody is not displayed.

Should you want to import implant kits in VIVO, there are two different ways:

- Import .IPFLIB
 - Create manually the implant kits
-


Import .IPFLIB

1. Log in as Administrator

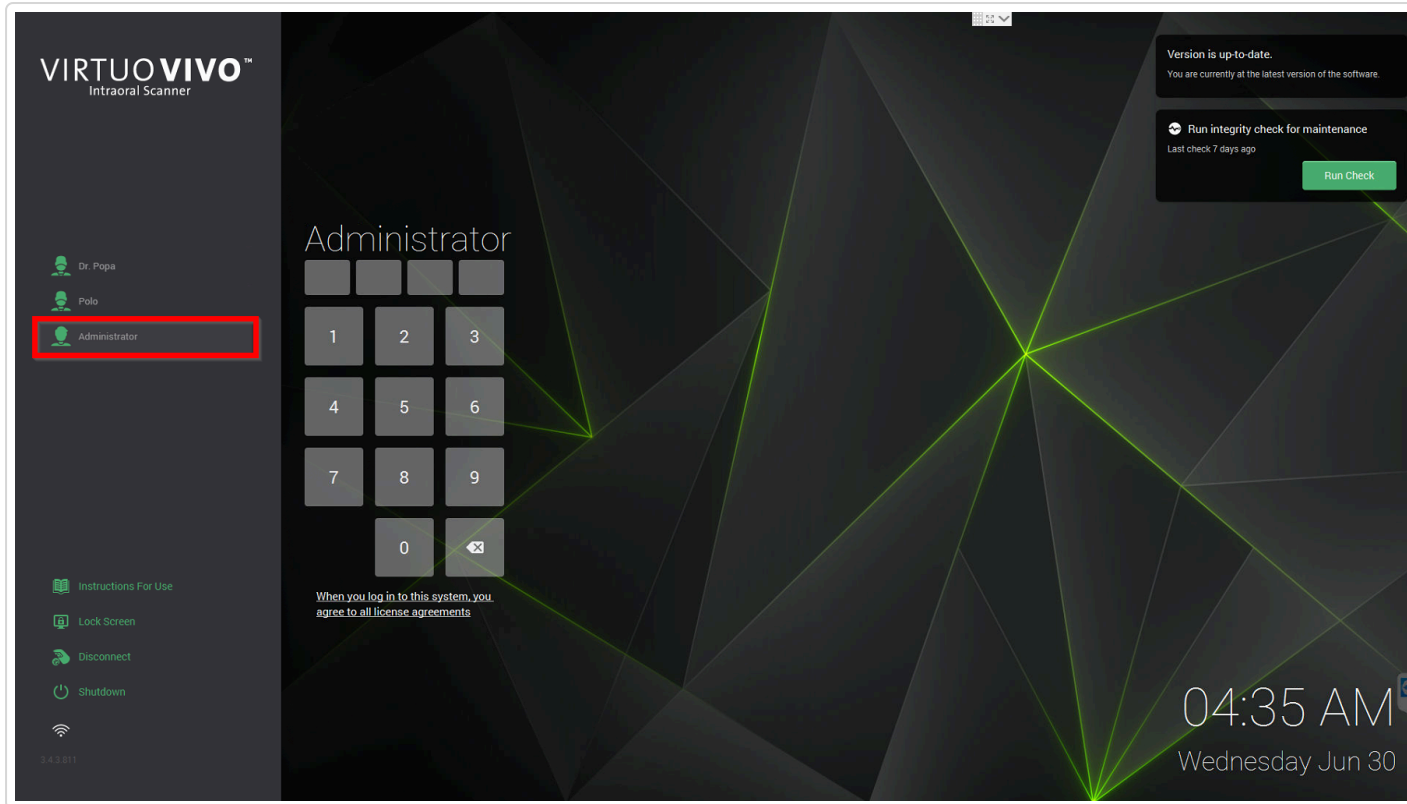


If the software version is 2.x you can use the password: 4two


2. Click on the Settings icon and then the "Implant kits"

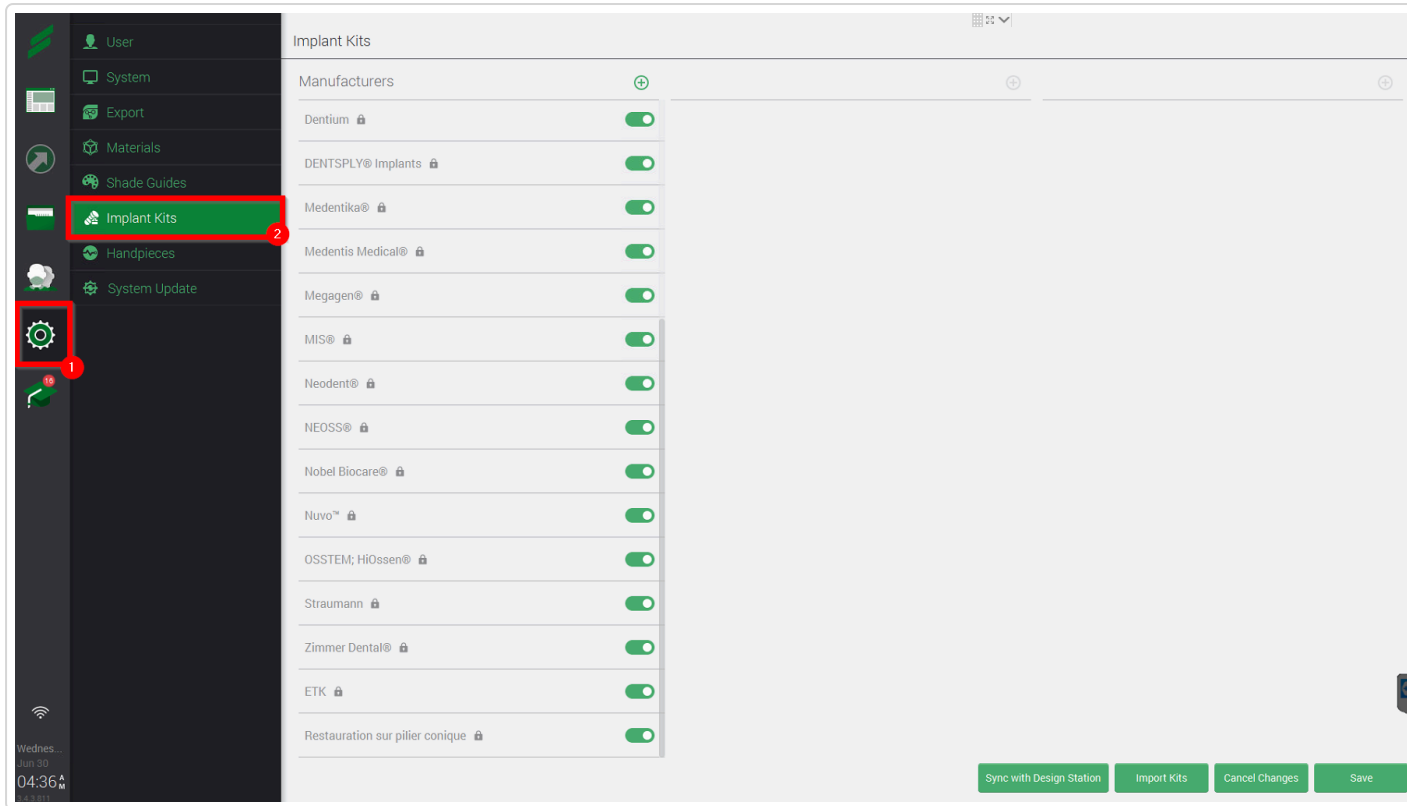

3. At the bottom right corner click on "Import Kits".

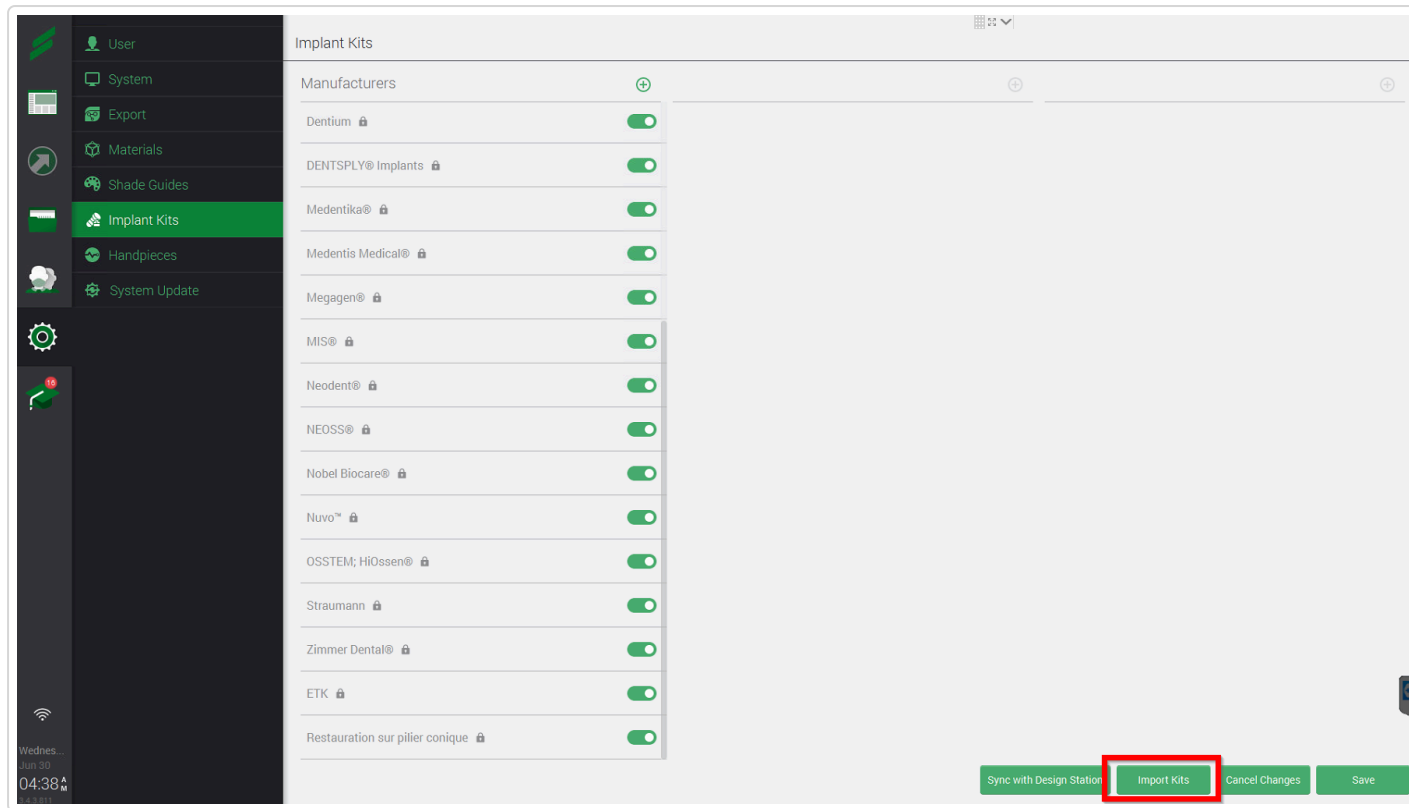

4. Go to the folder directory and Import all the .IPFLIB

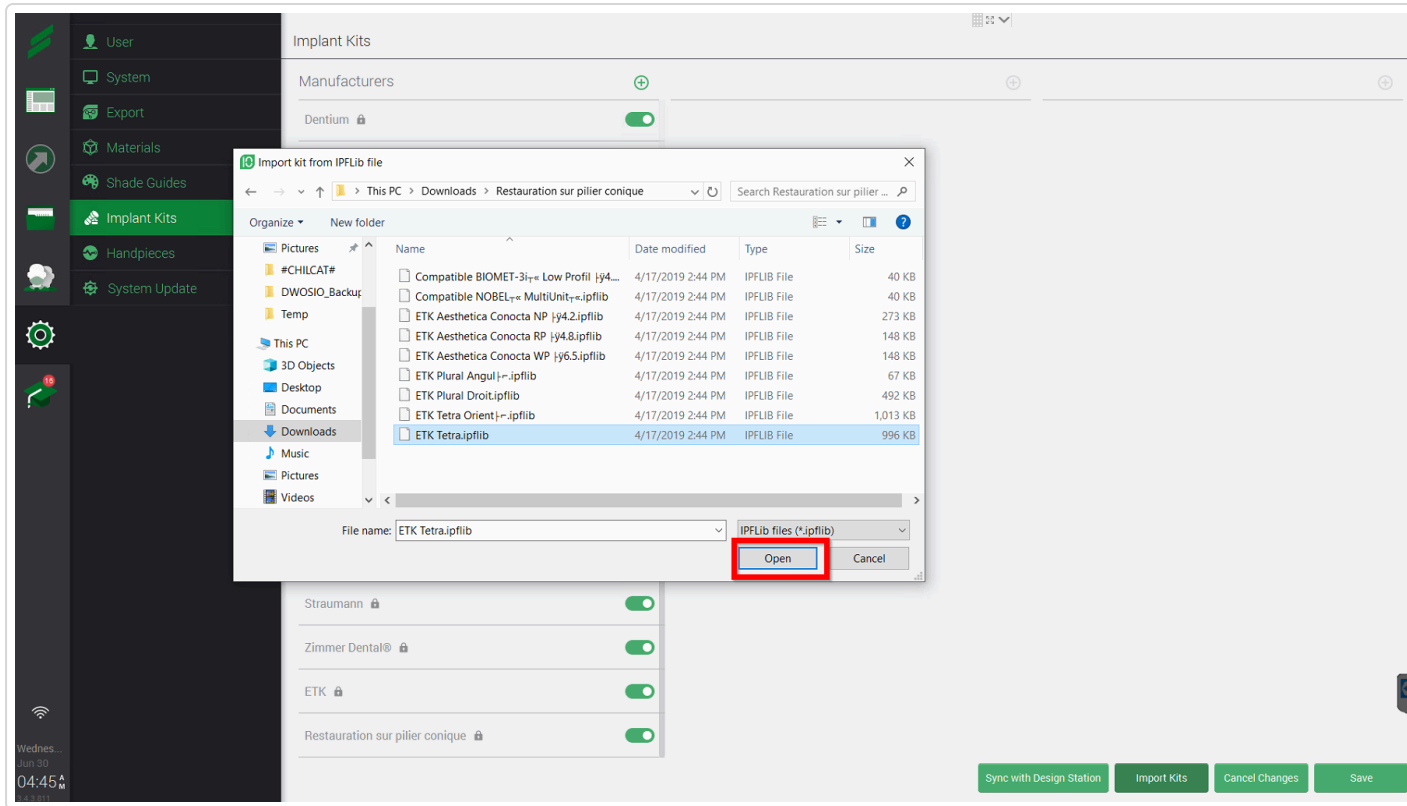


Nota Bene for Straumann Brand Implant Kit:


If the Implant kits are missing in the software, please go to C:\Program Files\Straumann CARES Intraoral Scanner\Software\data\implant\straumann

and select the whole content of the folder with CTRL + A


If it informs you about a conflict, select "Overwrite All".

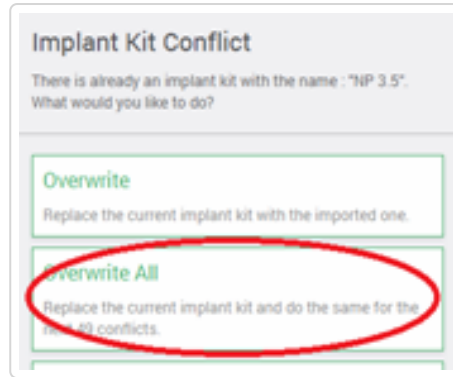


5. Press **“Save”**

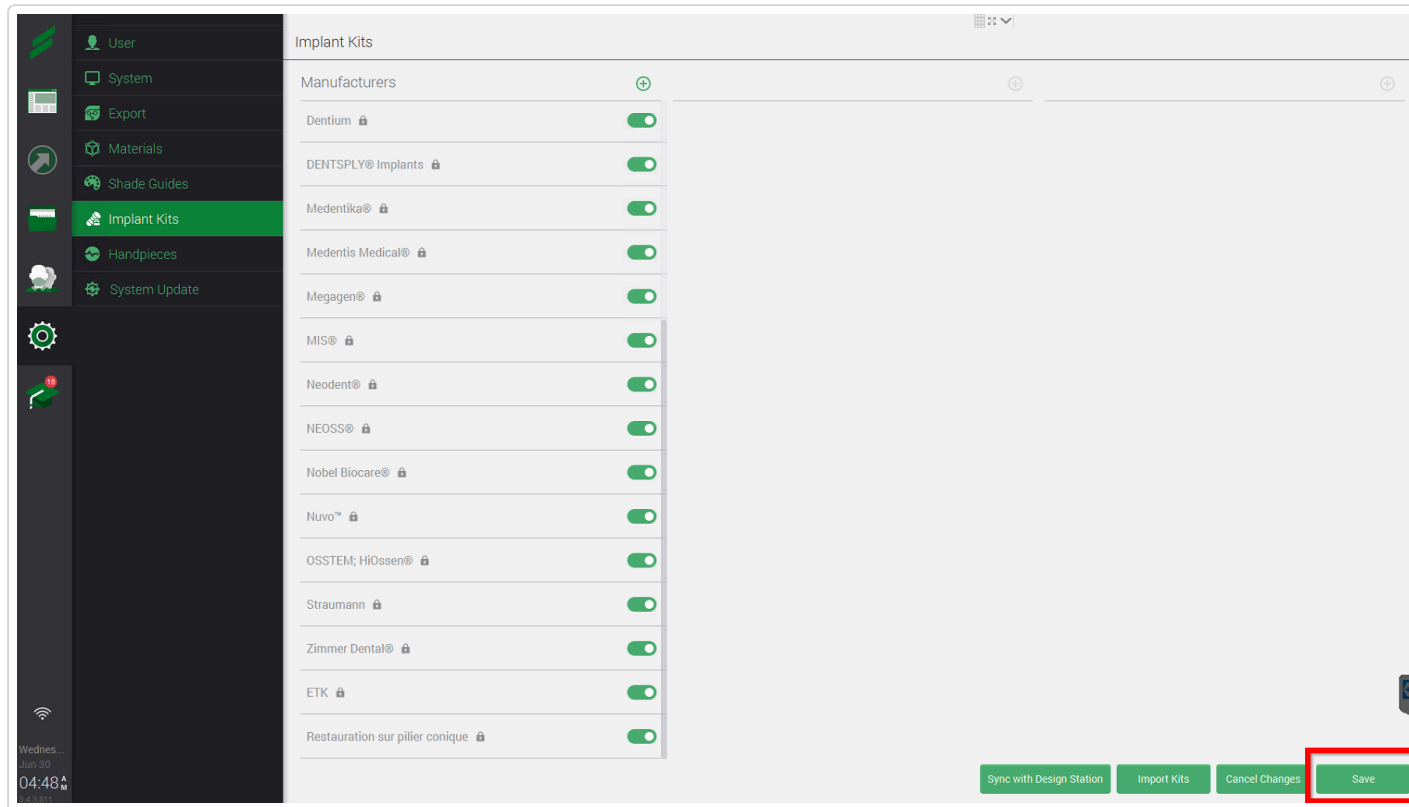


6. Log out of the Administrator account.

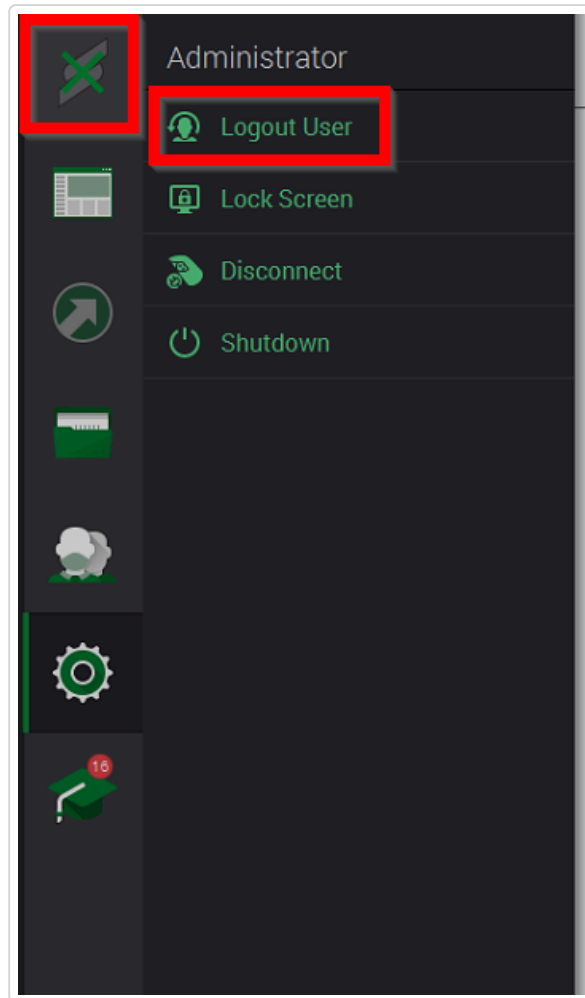

7. Login into the Dentist account.

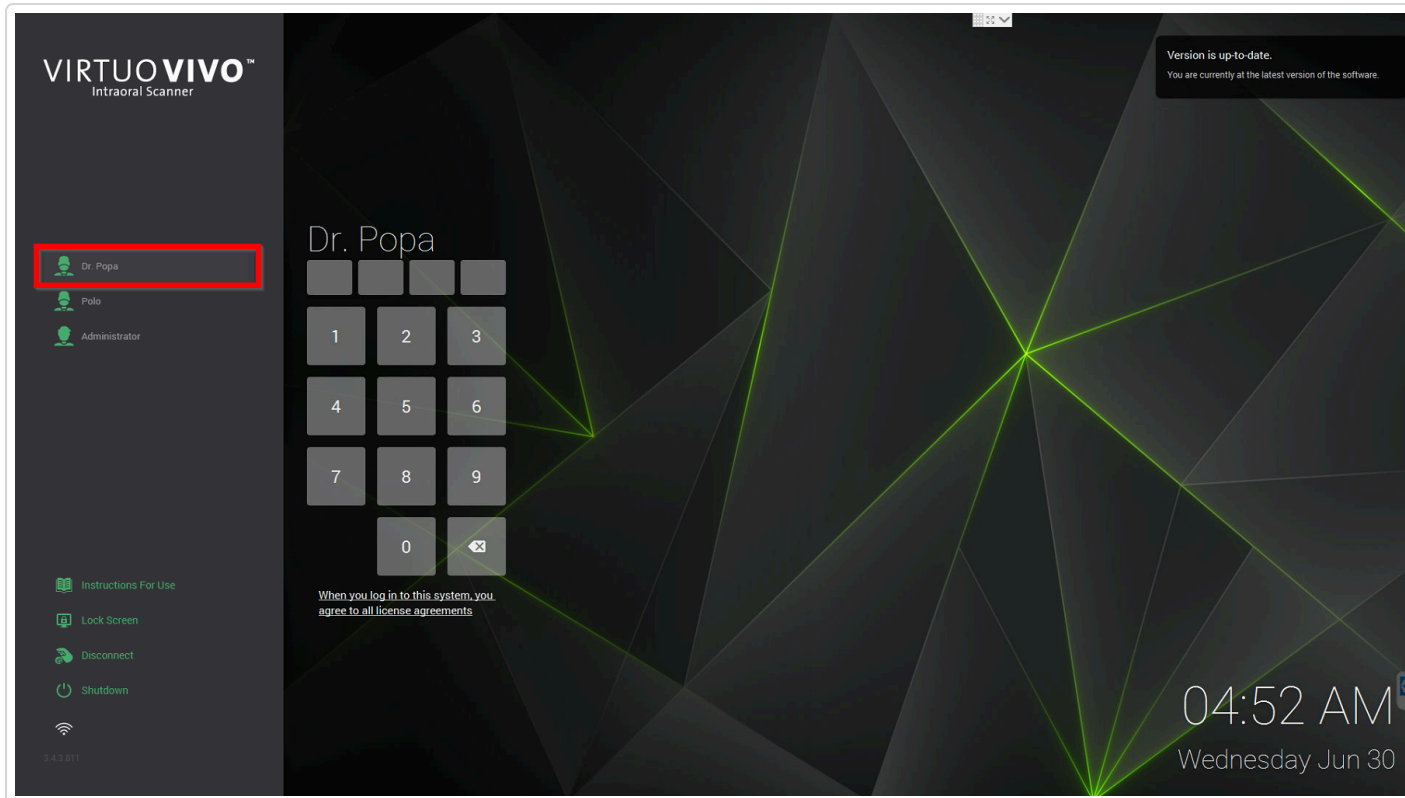


8. Click on the Settings icon and then the "Implant kits"
This step has to do be performed for each dentist account

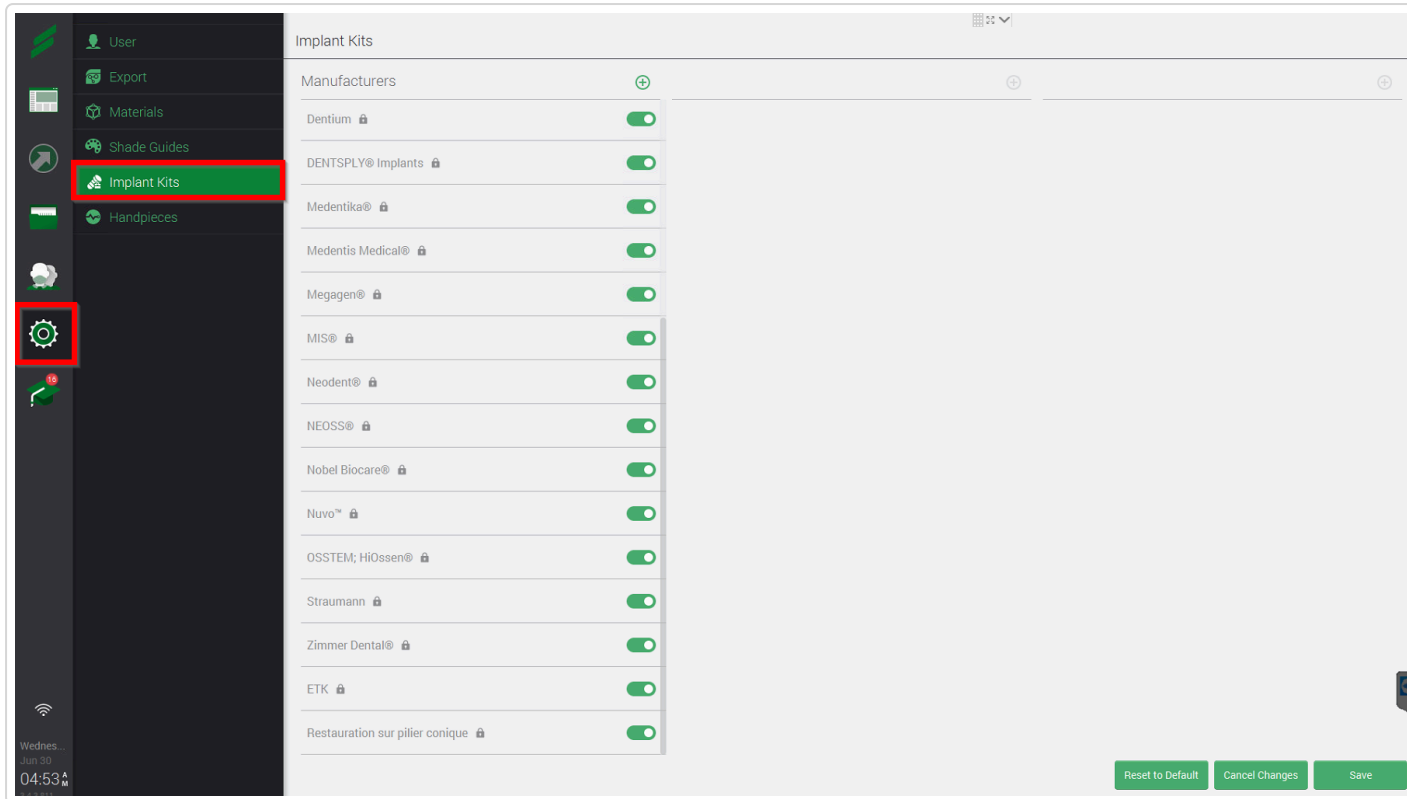


9. Press “ **Reset to Default**” and all the kits imported through Admin will appear. Press “**Save**”, log out, and continue with all other dentist users

This step has to do be performed for each dentist account

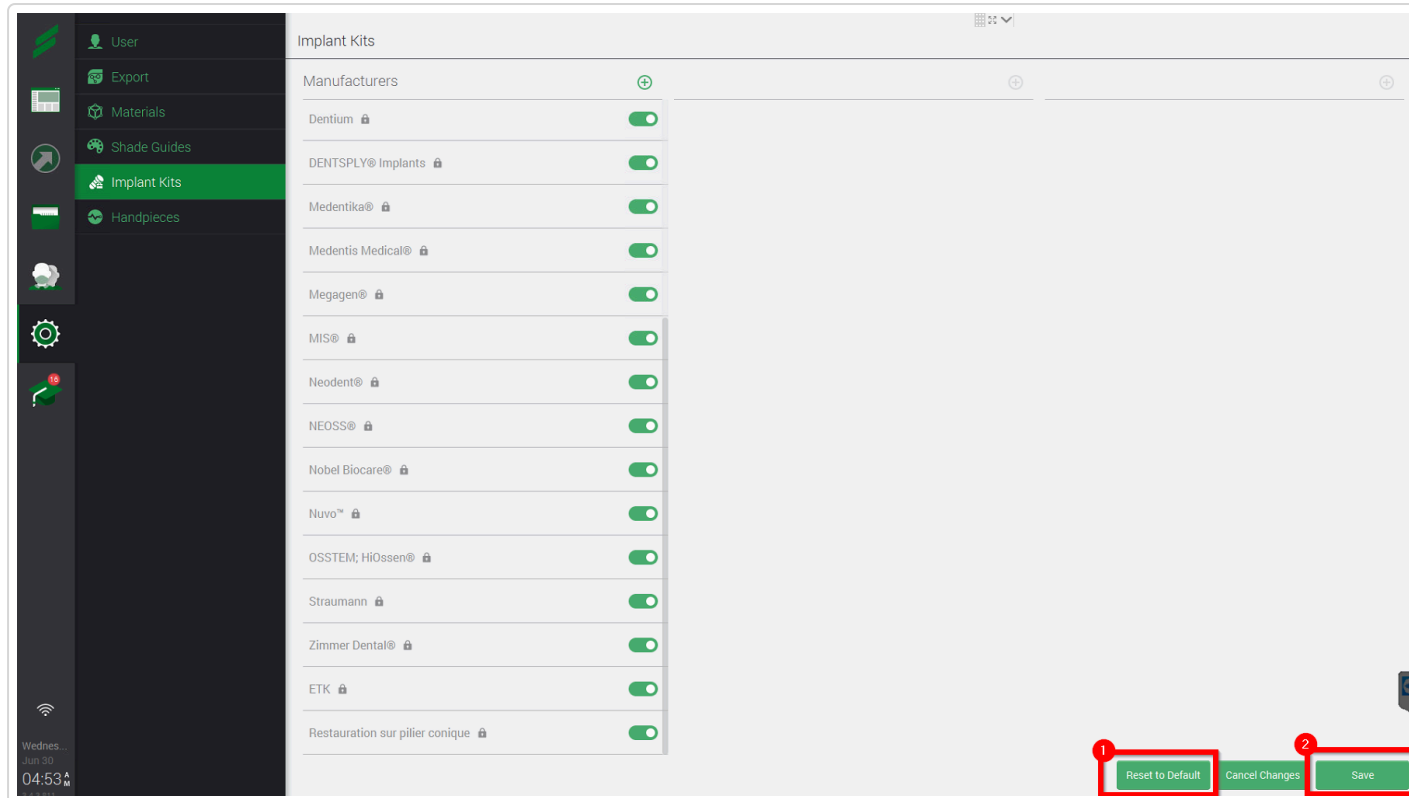

Create manually the implant kits

1. Log in as Administrator (if this username does not exist, look for a user with the administrator icon, red circle).

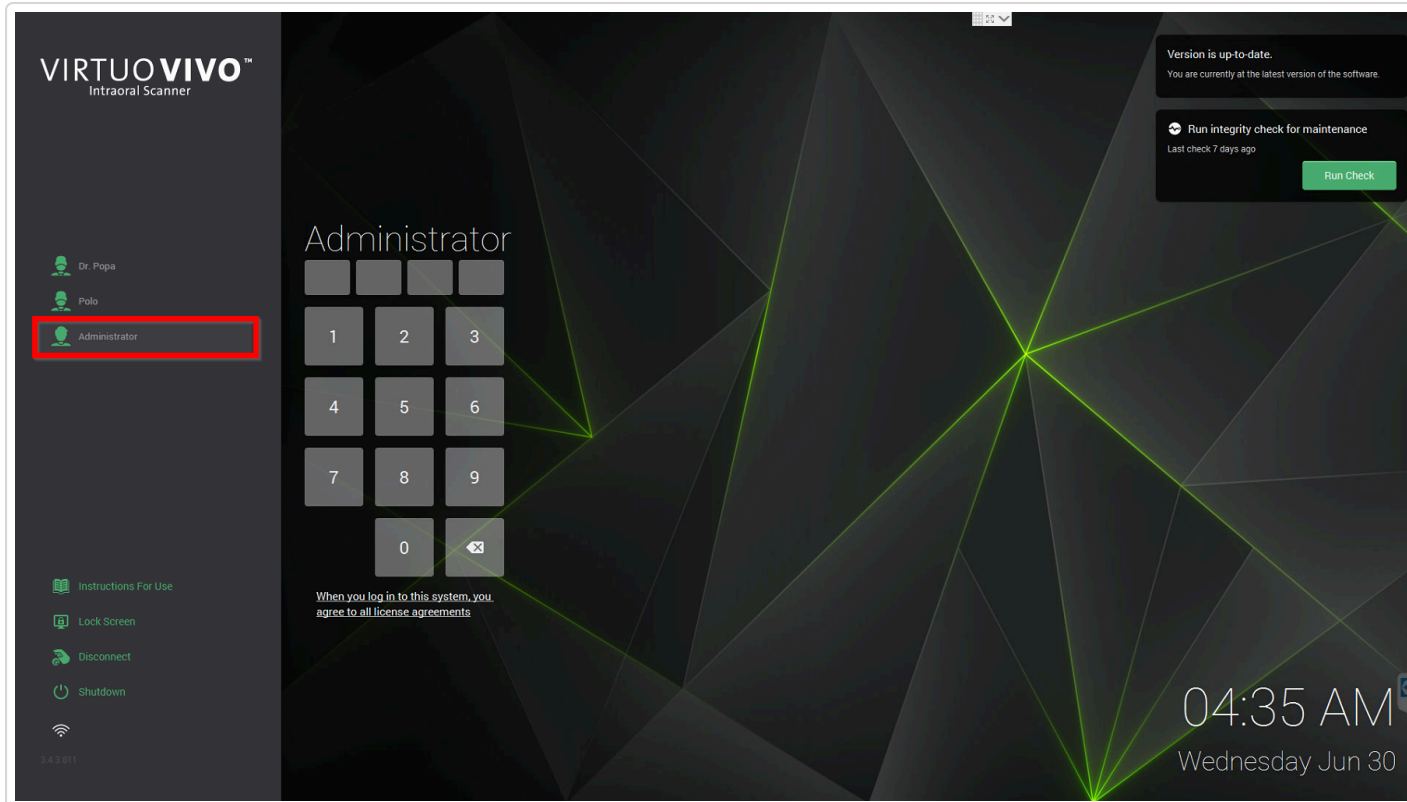


If the software version is 2.x you can use the password: 4two

2. Click on the Settings icon and then the "Implant kits"


The screenshot displays a software interface for managing implant kits. On the left, a dark sidebar contains navigation icons and labels: User, System, Export, Materials, Shade Guides, **Implant Kits** (highlighted with a red box and a '2' badge), Handpieces, and System Update. Below these is a gear icon (highlighted with a red box and a '1' badge) and a Wi-Fi icon. The main content area is titled 'Implant Kits' and features a list of manufacturers, each with a lock icon and a toggle switch. The manufacturers listed are: Dentium, DENTSPLY® Implants, Medentika®, Medentis Medical®, Megagen®, MIS®, Neodent®, NEOSS®, Nobel Biocare®, Nuvo™, OSSTEM; HIOssen®, Straumann, Zimmer Dental®, ETK, and Restauration sur pilier conique. At the bottom right of the main area, there are four buttons: 'Sync with Design Station', 'Import Kits', 'Cancel Changes', and 'Save'.

Manufacturer	Status
Dentium	On
DENTSPLY® Implants	On
Medentika®	On
Medentis Medical®	On
Megagen®	On
MIS®	On
Neodent®	On
NEOSS®	On
Nobel Biocare®	On
Nuvo™	On
OSSTEM; HIOssen®	On
Straumann	On
Zimmer Dental®	On
ETK	On
Restauration sur pilier conique	On

3. On the Manufacturer column, select +

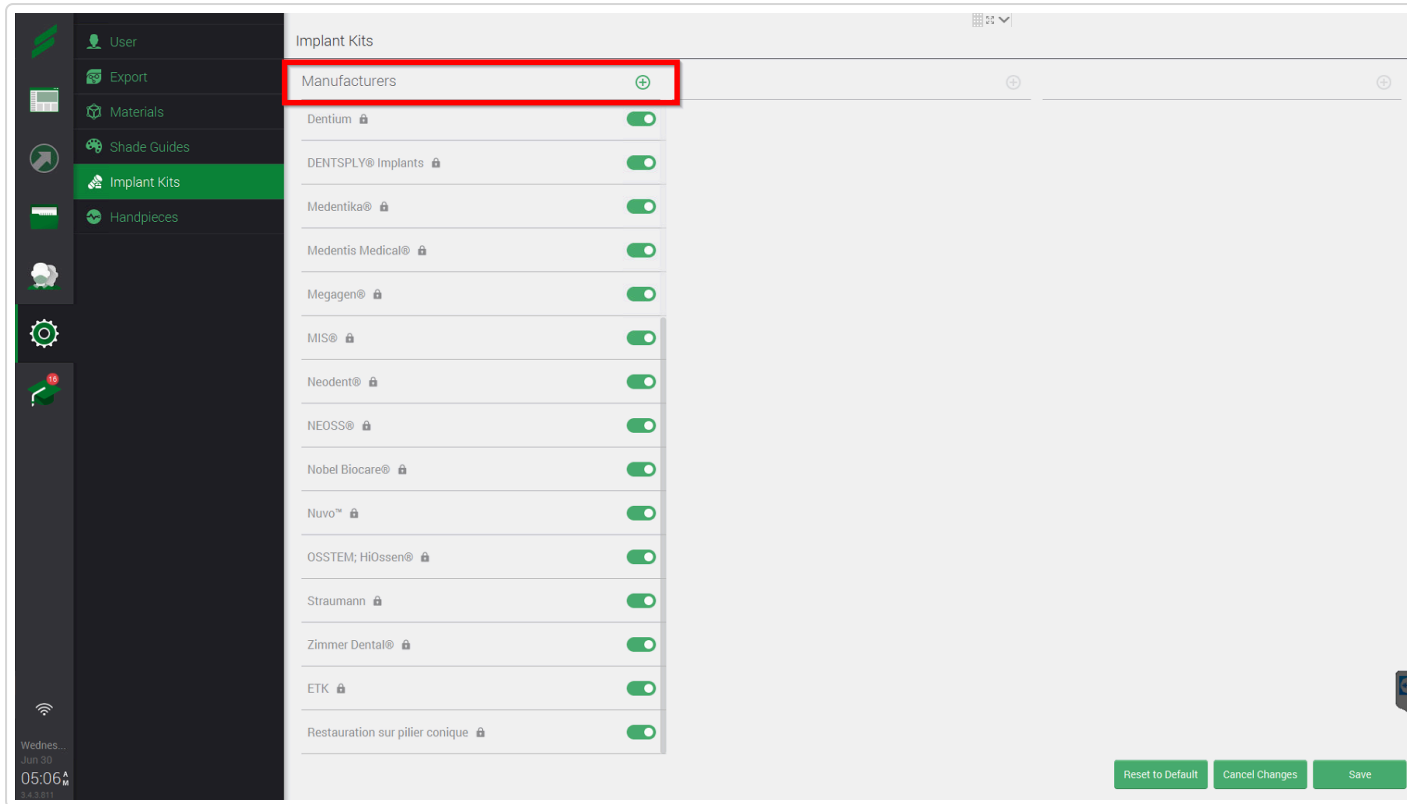

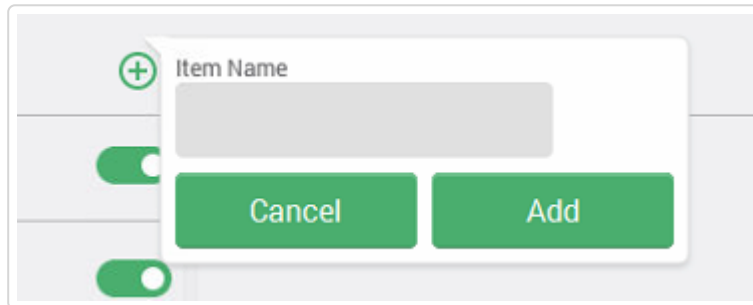


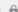
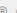

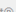

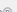





4. Fill in the Item name (Manufacturer) and press **Add**



5. Press + on the second column and Fill in the Item name. Press **Add**



Implant Kits

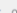Manufacturers

- Dentium 
- DENTSPLY® Implants 
- Medentika® 
- Medentis Medical® 
- Megagen® 
- MIS® 
- Neodent® 
- NEOSS® 
- Nobel Biocare® 
- Nuvo™ 
- OSSTEM; HiOssen® 
- Straumann 
- Zimmer Dental® 
- ETK 

Item Name

Cancel Add

  Manufacturer Name

Reset to Default Cancel Changes Save


6. Press **+** on the third column and Fill the Item name. Press **Add**

The screenshot shows the 'Implant Kits' management interface. On the left, there is a list of manufacturers with toggle switches. The main area displays a 'Sub-manufacturer' row. A modal form is open, showing an 'Item Name' input field and 'Cancel' and 'Add' buttons. The 'Add' button is highlighted with a red box.


7. As soon as you finish creating all the implant kits, Press on **Save**

The image shows a close-up of the bottom buttons: 'Reset to Default', 'Cancel Changes', and 'Save'. The 'Save' button is highlighted with a red box.


8. Log out of the Administrator account.

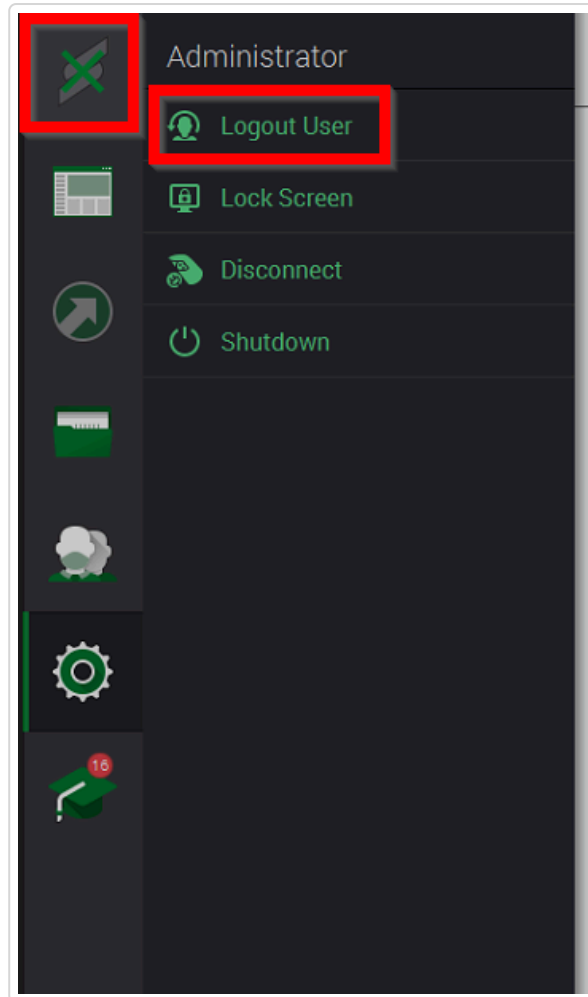

9. Login into the Dentist account.

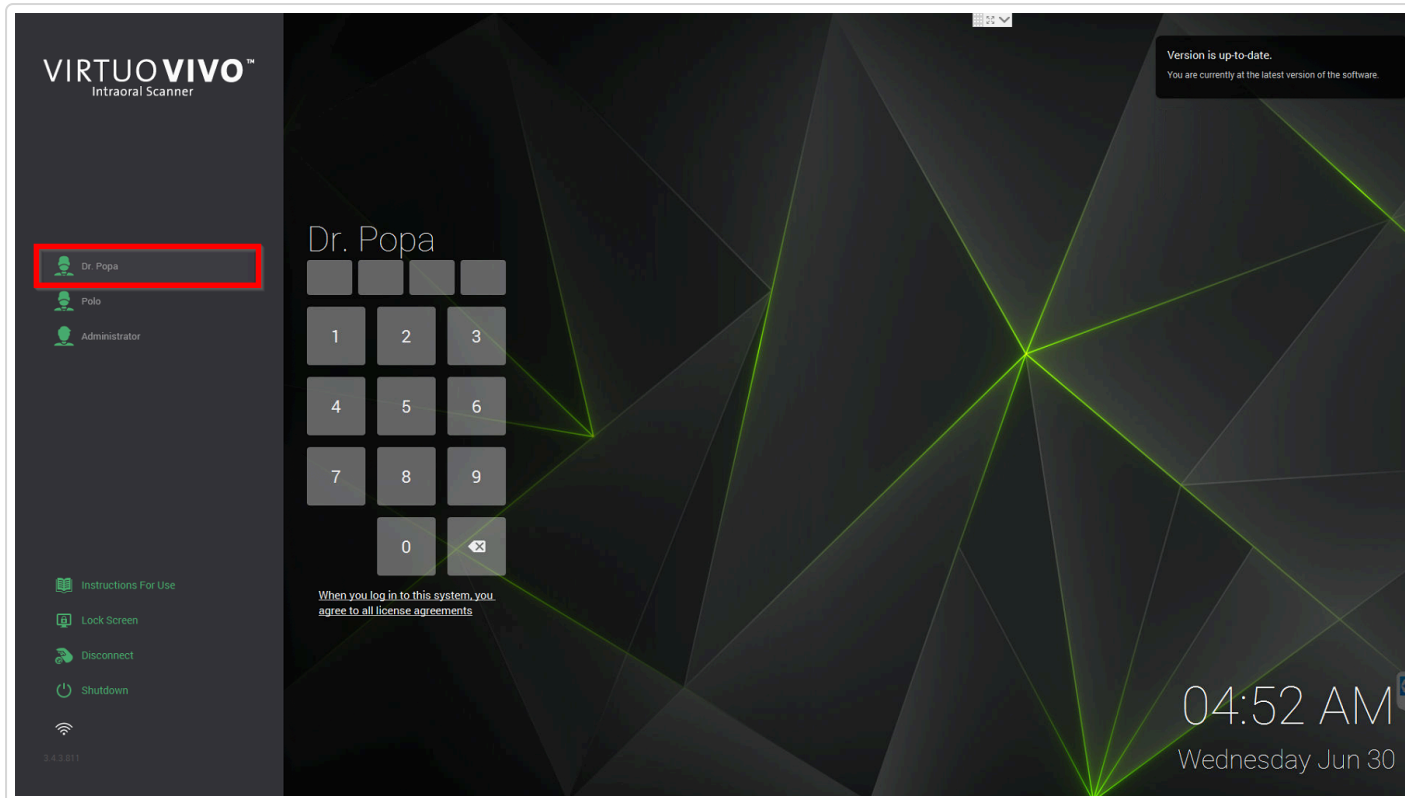

10. Click on the Settings icon and then the "Implant kits"
This step has to do be performed for each dentist account

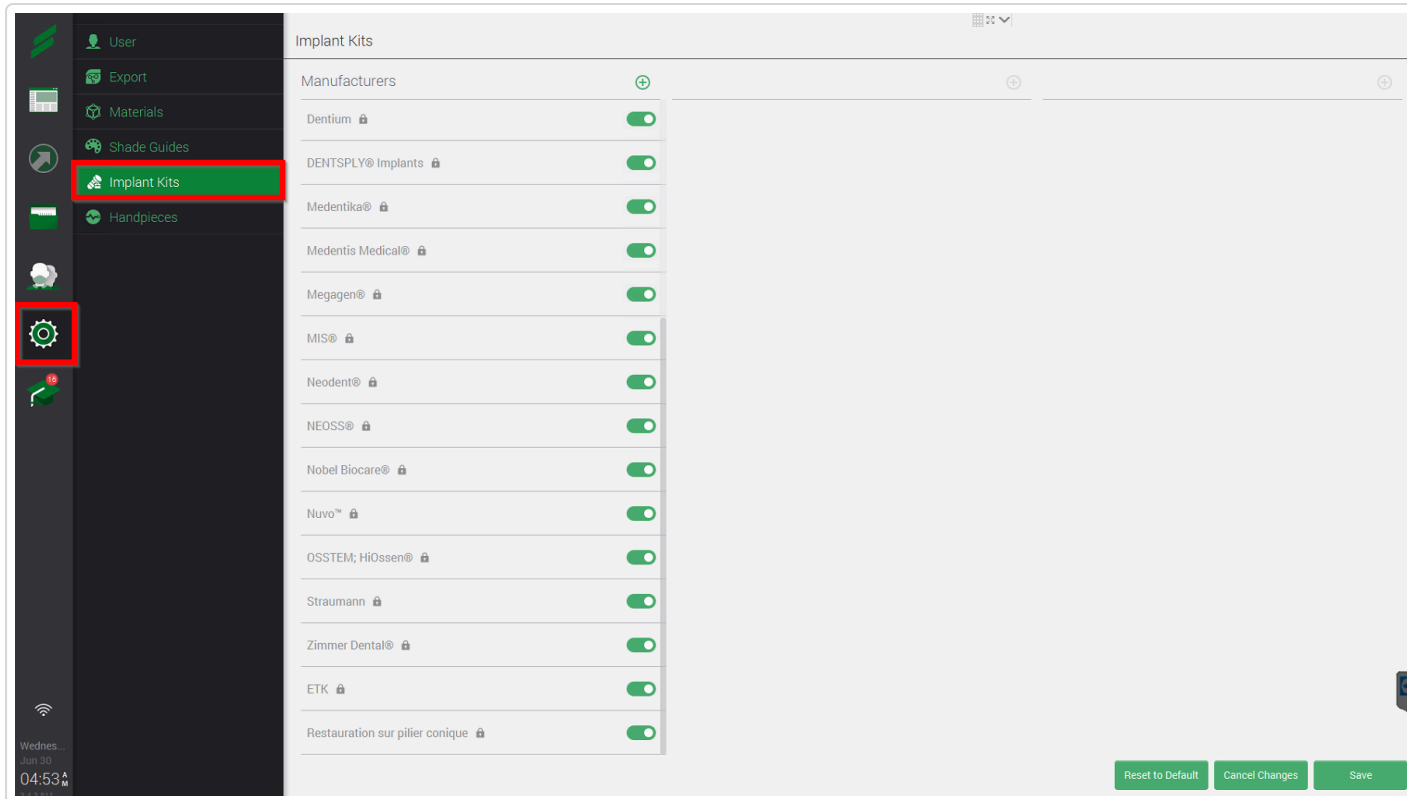

11. Press “ **Reset to Default**” and all the kits will appear. Press “**Save**”, log out, and continue with all other dentist users

This step has to do be performed for each dentist account

The screenshot displays the 'Implant Kits' configuration screen. On the left is a dark sidebar with navigation options: User, Export, Materials, Shade Guides, Implant Kits (highlighted), and Handpieces. The main area is titled 'Implant Kits' and contains a list of manufacturers, each with a lock icon and a green toggle switch. The manufacturers listed are: Dentium, DENTSPLY® Implants, Medentika®, Medentis Medical®, Megagen®, MIS®, Neodent®, NEOSS®, Nobel Biocare®, Nuvo™, OSSTEM; HiOssen®, Straumann, Zimmer Dental®, ETK, and Restauration sur pilier conique. At the bottom right, there are three buttons: 'Reset to Default' (marked with a red '1'), 'Cancel Changes', and 'Save' (marked with a red '2'). The system status bar at the bottom left shows 'Wednes... Jun 30 04:53 AM'.

Did this answer your question? 😊 ☹️

Last updated on January 23, 2023